

Power-Turn® ELITE

Lateral Rotation Mattress System with True Low Air Loss

POWER-TURN® ELITE MATTRESS SYSTEM - MODEL 9800

The Power-Turn® Elite Mattress System combines two clinically effective therapies: Lateral Rotation and True Low Air Loss in a portable system. The mattress utilizes 18 independent 9" deep air cells to provide proper pressure redistribution for the prevention and treatment of Stages I-IV pressure ulcers. Lateral Rotation Therapy reduces the risk of pulmonary complications for patients who are immobile. A Full, Left or Right Turn mode is offered with either a 25° Small Turn or a 40° Big Turn. The mattress replacement is designed to fit all standard hospital beds and includes a high density foam base, head bolster and abduction wedge for additional patient safety. The Power-Turn® Elite Lateral Rotation Mattress System supports patients weighing from 650 lbs. to 1000 lbs. The Power-Turn® Elite meets California Technical Bulletin #117 for fire retardancy. An optional fire barrier is available to meet the most stringent fire safety requirements when requested for foam base.

Ventilated low air loss cells have air holes throughout the mattress to help minimize skin maceration by reducing excess moisture under the patient. Air cells are constructed of TPU, thermal polyurethane material. This combination insures strength, durability and patient comfort.

Vyvox - III® - Urethane coated, multi-stretch, moisture-resistant / vapor-permeable, low shear, quilted cover, transfers moisture away from the patient's skin. The cover is washable, anti-microbial, fluid-resistant and fire-retardant.



Model 9800
Mattress and
Pump System

Head rest and leg abductor wedge provide support during the turning cycles. Head support is constructed from Viscotec®, a visco elastic foam. The leg abductor is constructed from PPF®, Pressurized Polymer Foam. Both are covered with Vyvox-III®.



Color coded hoses for easy connection.

The blower pump is controlled by a micro-processor which allows the caregiver total control of comfort and chosen therapy. Cycle time ranges from 5 to 95 minutes. The blower pump inflates the mattress within minutes. This reduces the waiting time for therapeutic treatment to begin.



 **Blue Chip**
Medical Products, Inc.

"Setting the Standard"

POWER-TURN®

SPECIFICATIONS

MODEL 9800 - POWER-TURN® ELITE MATTRESS REPLACEMENT & PUMP

Dimensions: 36" x 80" x 9"
Weight: 23 lbs.
Air Cells: 18 - 9" Air cells
 Ventilated low air loss bladders with air holes throughout mattress.

Side Bolster: 10" patented air side rail bolsters are built into mattress perimeter which supports the patient and insures safety during rotation cycles.

Degree of Turn: 25° Small Turn
 40° Big Turn

Mattress Cover: Vyvex-III® - Urethane coated, multi-stretch, low shear, moisture vapor permeable, quilted, removable, zippered cover, is anti-microbial and meets Cal. Tech. #117 for fire-retardancy.



insert illustrates support bladders and air side rails

MODEL 9801 - POWER-TURN® ELITE PUMP ONLY

Dimensions: 17L" x 7W" x 10.5H"
Weight: 12 lbs.
Electrical: 120 60 hz
Listings: UL CE S
Cycle Time: 5 - 95 Minutes
Air Flow: 1800 Liters per minute
Ground Resistance: 0.1 ohms maximum
Current: 5A
Power Cord: 12 ft. cord with hospital grade plug
Modes of Operation: Auto full-turn, left turn, right turn, auto-firm and static.



ORDERING INFORMATION

Model 9800	Power-Turn® Elite	650 lbs Capacity	36" x 80" x 9"
Model 9842	Power-Turn® Elite	800 lbs Capacity	42" x 80" x 9"
Model 9848	Power-Turn® Elite	1000 lbs Capacity	48" x 80" x 9"
Model 9800-AW	Power-Turn® Elite Abductor Wedge		
Model 9800-HR	Power-Turn® Elite Head Rest		
Model 9826	Replacement zippered cover		

Power Turn® Elite meets Group 2 Support Surface requirements for Medicare E0277.
 Patient must meet all medical criteria to qualify for reimbursement.
 Use of side rails is recommended for patient safety.

WARRANTY

Pumps: 1 year limited
Mattress: 1 year limited
Cover: 6 months limited

PDAC Approved
 Medicare E0277



7-11 Suffern Place, Suffern NY 10901

Tel: (845) 369-7535 • Fax: (845) 369-7633 • Toll Free: (800) 795-6115

www.bluechipmedical.com

"Setting the Standard"

©BLUE Chip Medical Products, Inc. 2018

Alternating Pressure Mattresses
 Alternating Pressure Pads & Pumps
 Low Air-Loss Mattress Systems
 Lateral Rotation Mattress Systems
 Gel Mattresses
 Gel Mattress Overlays
 Static-Air Mattress Overlays
 Water Mattress Overlays
 Wheelchair Cushions
 Patient Lift Slings
 Side Rail Bumpers
 Cart Covers
 OR Replacement Table Pads
 Replacement Stretcher Pads
 Cubicle Curtains
 Hospital Pillows