

FILLING INSTRUCTIONS FOR BLUE CHIP MEDICAL 5500 STAT-H2O 5700 STAT-GEL

- Open the Fill Cap located on the right side foot section of the mattress
- Lift the Fill Cap till it pops up for easy filling
- Empty the ENTIRE contents of Gel Powder
- Evenly distribute the powder throughout the mattress
- Lay the mattress flat and start to fill with water
- Lift the bottom of mattress to the fold line and fill to the appropriate level
- Remove excess air by opening small air valve at bottom of mattress
- Fine adjustments can be made once the patient is placed on the mattress
- Gel will start to coagulate shortly

