

GRZ[®] Wheelchair Cushion

Self Adjusting Air Pressure Wheelchair Cushion

GRZ[®] WHEELCHAIR CUSHION

The GRZ is dynamic adjustable air bladder system consisting of 3 individual air cell chambers which redistribute pressure and adjust to body weight and position. Ideal for patients with moderate to high risk of skin breakdown who require significant postural support. The GRZ allows the therapist to offer correct posture, proper alignment and trunk stability as well as adjustments for pelvic obliquities. Does not require pump for adjustment. Adjustable strap secures cushion to wheelchair.



Shown without cover


3 independent air cell chambers



Self Adjusting Air Valve

HOW TO USE THE GRZ CUSHION:

- Open all three valves to allow the three chambers to fill with air.
- Close off valves
- Have patient sit on the cushion
- Open valves individually until the optimum internal pressure is achieved
- Close off valves
- In order to reset cushion, remove the patient from the cushion and open all three valves
- Close off valves and start from the beginning

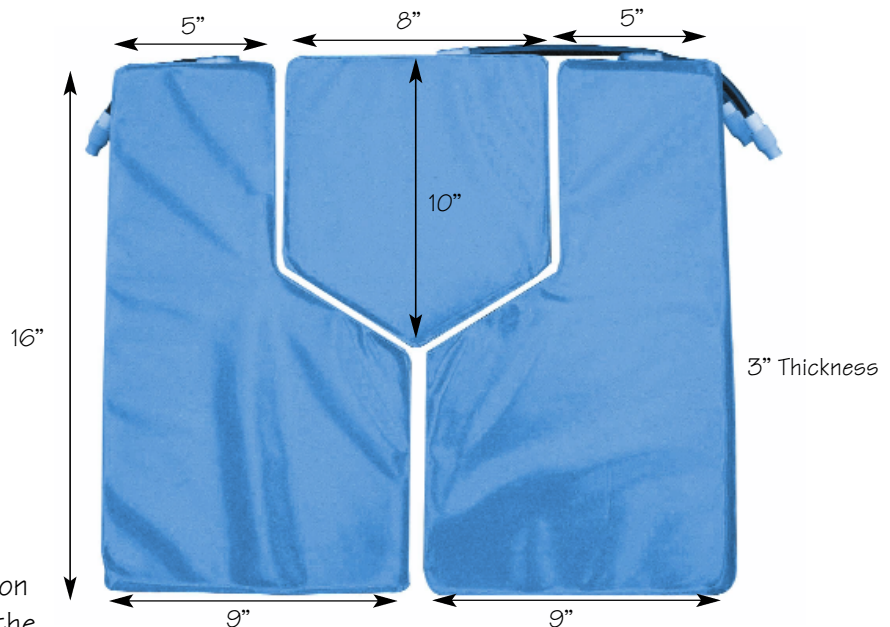

3" Thickness

HOW TO CLEAN THE GRZ CUSHION: Wipe the cushion down between patients using a damp soft cloth and mild detergent. Covers can be washed in a washing machine. Tumble Drying is not recommended.



Blue Chip

Medical Products, Inc.

"Setting the Standard"