

EZ-TURN™

Patient Turning & Repositioning Device

EZ-TURN™ - PATIENT POSITIONING DEVICE

Turning is essential for patient care, pressure redistribution and to help reduce pressure injury. When the patient is unable to turn without assistance, the caregiver and patient are at greater risk of injury.



EASY, SAFE & COMFORTABLE PATIENT REPOSITIONING

EZ-Turn™ off-loads the patient's weight while the rounded shape and turning handles give caregivers the control to easily and safely reposition the patient.

EZ-Turn™ provides patient comfort and security and can help prevent injury for both the patient and caregiver.

FEATURES

- Core is constructed from high-density CertiPUR-US® certified, PPF™ foam
- Foam is latex-free and fire retardant
- Durable and resilient for years of use
- Covered in a durable heavy-duty anti-microbial medical grade vinyl
- Double stitched to improve performance and longevity
- Heavy-duty double stitched handles for excellent grasp and perfect control
- Cleans easily
- Made in the USA

EZ-TURN™

SPECIFICATIONS

Model BC-EZTA	Standard 18"w x 16"h x 5"d (leg opening 4.5"w)
Model BC-EZTA-BAR	Bariatric 22"w x 18.5"h x 7"d (leg opening 6"w)
Cover:	Medical Grade Vinyl, Double Stitched
Core:	Certipur certified, PPF™ foam



Model Shown BC-EZTA

TURN EASY, TURN SAFELY

With the patient lying on their back in bed:

STEP 1

Position the EZ-Turn™ flat on the bed with the leg supports facing down toward the foot of the bed. Position the patient's left and right leg over each leg opening.



STEP 2

Stand the EZ-Turn™ upright so the patient's legs fit through the leg opening.



STEP 3

With the patient's legs properly positioned, push or pull the side handles to easily roll the patient.



WARRANTY

6 months limited warranty on foam core



Blue Chip

Medical Products, Inc.

7-11 Suffern Place, Suffern NY 10901

Tel: (845) 369-7535 • Fax: (845) 369-7633 • Toll Free: (800) 795-6115

www.bluechipmedical.com

"Setting the Standard"

©Blue Chip Medical Products, Inc. 2021

Alternating Pressure Mattresses
Alternating Pressure Pads & Pumps
Low Air-Loss Mattress Systems
Lateral Rotation Mattress Systems
Gel Mattresses
Gel Mattress Overlays
Static-Air Mattress Overlays
Water Mattress Overlays
Wheelchair Cushions
Patient Lift Slings
Side Rail Bumpers
Cart Covers
OR Replacement Table Pads
Replacement Stretcher Pads
Cubicle Curtains
Hospital Pillows