

Adapt Pro^{Elite™}

Alternating Pressure Enhanced Low Air Loss Mattress System with Patient Sensing Technology

MODEL 9200 - ADAPT PRO ELITE™ WITH PATIENT SENSING TECHNOLOGY/IMMERSION THERAPY

The Adapt Pro Elite™ Alternating Pressure Mattress System is the latest technology for the customized treatment and prevention of stages I-IV pressure injury. The Adapt Pro Elite™ offers advanced therapeutic features: Low Air Loss incorporated inside the unique cover enhances therapy and controls micro-climate. Automatic two-stage fowler function helps to prevent sliding & skin shear. Patient Sensing Technology (PST) monitors and measures the patient and automatically adjusts every two hours to create the patient's optimal therapy and optional Immersion Therapy.

Designed to fit all standard hospital beds, Adapt Pro Elite™ is available in 36", 42" & 48" and supports patient weights up to 400 lbs., 650 lbs., & 800 lbs. The user friendly controls and ease of setup make Adapt Pro Elite™ ideal for institutional use and the home-bound patient.

PUMP CONTROL MODES & FEATURES

Adapt Pro Elite's dual compressor pump is feature rich, whisper quiet and extremely durable.

MODES

- Alternation
- Static
- Static Care - Falls back to Alternating mode in 20 mins.
- Pulsation - Immersion Therapy (optional function)

PATIENT SENSING TECHNOLOGY (PST)

- Auto Detection (PST) monitors patient & adjusts every two hours.
- 2-Stage Fowler Function - Adjusts pressures in the sacral area based on patient position. Incline 30° (LL) increases 10%, 40° (LH) incline increases by additional 5%. Reduces sliding and skin shear.

PUMP FEATURES

- Patient Comfort Control
- Cycle Time 10 - 30 minutes
- Auto-Firm with Fall-back
- Lock-Out Mode
- Easy-to-read digital display
- Low Pressure & Power Failure Alarms
- Heavy duty dual-compressor

DURABLE DUAL BLADDERS DESIGN

20 independent 8" deep air cells is constructed of (TPU) thermal polyurethane, laminated to a medical grade nylon for maximum durability and performance. The dual bladder design maintains air and prevents patients from bottoming out during power outages.

VYVEX-I® ENHANCED LOW AIR LOSS COVER

Low Air Loss therapy is incorporated inside the mattress cover, transferring moisture away from the patient's skin and maintaining the optimal micro-climate. Vyvex-I® is a low shear, low friction nylon cover fabric that is washable, anti-microbial, vapor permeable, fluid resistant and fire retardant. Cover is loose fitting and will conform to the patient.



Shown: Model 9200



Shown: Model 9200SRS
with optional side rail safety bolster



CPR

Bolster Release

CPR & Bolster Release valves quickly deflate mattress system for medical emergencies.



Optional Electric Inflator/Deflator

ADAPT PRO ELITE™

SPECIFICATIONS

MATTRESS

Dimensions:	36" x 80" x 8", 42" x 80" x 8", 48" x 80" x 8"
Supports Weight:	400 lbs., 650 lbs., 800 lbs.
Air Cells:	20, 8" TPU Nylon Air cells 3 Static cells in head section
Low Air Loss Cover:	Vyvox-1® removable zippered, low shear, moisture resistant / vapor permeable cover



Adjustable anchor straps secures mattress to bed frame. Fire evacuation handles standard

PUMP

Dimensions:	16.5" x 9" x 5"
Weight:	10 lbs.
Electrical:	120V 60Hz
Cycle Time:	10 - 30 minutes
Air Flow:	9 + 9 = 18 LPM
Listings:	UL CE SPS
Ground Resistance:	0.1 ohms maximum
Current Usage:	less than 100 micro amps
Power Cord:	16.5 ft.
Construction:	ABS fire retardant UL 94V-0



ADAPT PRO ELITE™ ORDERING INFORMATION

When ordering mattresses specify with or without SRS Side Rail Safety Bolster

Model: 9200SR5	36" Mattress with Side Bolsters & Pump Control Unit
Model: 9200SR5-42	42" Mattress with Side Bolsters & Pump Control Unit
Model: 9200SR5-48	48" Mattress with Side Bolsters & Pump Control Unit
Model: 9200	36" Mattress without Side Bolsters & Pump Control Unit
Model: 9200-42	42" Mattress without Side Bolsters & Pump Control Unit
Model: 9200-48	48" Mattress without Side Bolsters & Pump Control Unit
Model: 9201	Pump Control Unit for 36" Mattress
Model: 9201-42	Pump Control Unit for 42" Mattress
Model: 9201-48	Pump Control Unit for 48" Mattress
Model: 9220SR5	36" Mattress only with Side Bolsters
Model: 9220SR5-42	42" Mattress only with Side Bolsters
Model: 9220SR5-48	48" Mattress only with Side Bolsters
Model: 9202	36" Mattress only without Side Bolsters
Model: 9202-42	42" Mattress only without Side Bolsters
Model: 9202-48	48" Mattress only without Side Bolsters
Model: 9623	Electric Inflator & Deflator

Adapt Pro Elite™ meets Group 2 Support Surfaces requirements for Medicare E0277.
Patient must meet all medical criteria to qualify for reimbursement.

WARRANTY

Mattress & Pump	1 year limited
Cover:	6 months limited



7-11 Suffern Place, Suffern NY 10901

Tel: (845) 369-7535 • Fax: (845) 369-7633 • Toll Free: (800) 795-6115

www.bluechipmedical.com

"Setting the Standard"

©Blue Chip Medical Products, Inc. 2022

Alternating Pressure Mattresses
Alternating Pressure Pads & Pumps
Low Air-Loss Mattress Systems
Lateral Rotation Mattress Systems
Gel Mattresses
Gel Mattress Overlays
Static-Air Mattress Overlays
Water Mattress Overlays
Wheelchair Cushions
Patient Lift Slings
Side Rail Bumpers
Cart Covers
OR Replacement Table Pads
Replacement Stretcher Pads
Cubicle Curtains
Hospital Pillows