

# Apollo 3-Port™

## *Alternating Pressure Mattress System with Low Air Loss*


### MODEL 4600 - APOLLO 3-PORT™

The Apollo 3-Port™ Alternating Pressure Mattress System with Low Air Loss incorporates the features that clinicians demand for their patients in a portable system. The mattress utilizes 16 independent 8" deep air cells to provide pressure redistribution for the prevention and treatment of Stages I-IV pressure ulcers. User friendly controls and ease of setup make Apollo - 3 Port™ ideal for institutional use and the home-bound patient. The mattress is designed to fit all standard hospital beds. Apollo 3-Port™ support patient weights up to 350 lbs. Apollo 3-Port™ meets California Technical Bulletin #117 for fire retardancy. An optional fire barrier is available to meet the most stringent fire safety requirements when requested.



Vyvox-III™ multi-directional stretch fabric transfers moisture away from the patient's skin. The cover is low shear, washable, anti-microbial, fluid resistant and fire retardant. Cover is loose fitting and will conform to the patient.

CPR Valve quickly deflates mattress system for medical emergencies. Optional electric pump available.

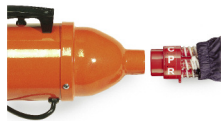

Ventilated bladders include 40 air holes in sacral and torso areas to help minimize skin maceration by reducing excess moisture under patient. Air cells are constructed of (TPU) thermal polyurethane, laminated to a medical grade nylon. Dual bladder design prevents patients from bottoming out, while sitting up and will maintain air during power outages.



Optional 4 bladder heel section provides advanced dynamic therapy specific to the prevention and treatment of heel ulcers.

Shown with optional  
4600-SRS  
Side Rail Safety System



### PUMP CONTROL FEATURES

- Unique **3-Port** design provides continuous air flow to the 3" static air base of mattress.
- Alternating mode offers the benefit of alternating pressure therapy and reactive hyperemia. This helps to redistribute pressure well below 32mmHg.
- Control dial adjusts mattress from soft to firm for patient comfort.
- Normal Pressure Alert indicates that system is operating correctly.
- Low Pressure Alert indicates that output needs to be increased for heavier patients or accidental disconnection of hoses.

**Blue Chip**

Medical Products, Inc.

*"Setting the Standard"*



# APOLLO 3-Port™

## SPECIFICATIONS

### MATTRESS ONLY

Dimensions:	35" x 80" x 8"
Weight:	18 lbs.
Air Cells:	16, 8" Air cells 13 Alternating cells 10 Ventilated cells with 40 air holes 3 Static cells in head section
Air Cells Option:	14, 8" bladders & 4, 4" foot section air bladders
Cover:	Vyvex-III™ removable zippered, low shear, moisture resistant / vapor permeable cover



Adjustable anchor straps secure mattress to bed frame. Fire evaluation handles standard

### PUMP

Dimensions:	10.75" x 7.5" x 3.5"
Weight:	6.8 lbs.
Electrical:	120V 60Hz
Cycle Time:	10 minutes
Air Flow:	7.5 liters
Listings:	UL CE S
Ground Resistance:	0.1 ohms maximum
Current Usage:	less than 100 micro amps
Power Cord:	12 ft.
Construction:	ABS fire retardant UL 94V-0



## APOLLO 3-PORT™ ORDERING INFORMATION

Model: 4600	Mattress & Pump Control Unit
Model: 4600-SRS	Mattress & Pump Control Unit w/ Side Rail System
Model: 4610	Pump Control Unit only
Model: 4620	Mattress only
Model: 4622	Carry Bag with Strap
Model: 9623	Electric Inflator & Deflator
Model: 4624	Replacement 8" Ventilated Bladders
Model: 4625	Replacement 8" Cell Bladders
Model: 4626	Replacement Zippered Cover
Model: 4626-SRS	Replacement SRS Safety Rail System

Apollo 3-Port™ meets Group 2 Support Surfaces requirements for Medicare E0277.  
Patient must meet all medical criteria to qualify for reimbursement.

### WARRANTY

Power Control Unit:	1 year limited
Mattress:	1 year limited
Cover:	6 months limited

PDAC Approved  
Medicare E0277



7-11 Suffern Place, Suffern NY 10901

Tel: (845) 369-7535 • Fax: (845) 369-7633 • Toll Free: (800) 795-6115

[www.bluechipmedical.com](http://www.bluechipmedical.com)

*"Setting the Standard"*

©Blue Chip Medical Products, Inc. 2014

Alternating Pressure Mattresses  
Alternating Pressure Pads & Pumps  
Low Air-Loss Mattress Systems  
Lateral Rotation Mattress Systems  
Gel Mattresses  
Gel Mattress Overlays  
Static-Air Mattress Overlays  
Water Mattress Overlays  
Wheelchair Cushions  
Patient Lift Slings  
Side Rail Bumpers  
Cart Covers  
OR Replacement Table Pads  
Replacement Stretcher Pads  
Cubicle Curtains  
Hospital Pillows