

# EZ-TURN™

## Patient Turning & Repositioning Device

### EZ-TURN™ - PATIENT POSITIONING DEVICE

Turning is essential for patient care, pressure redistribution and to help reduce pressure injury. When the patient is unable to turn without assistance, the caregiver and patient are at greater risk of injury.



### EASY, SAFE & COMFORTABLE PATIENT REPOSITIONING

EZ-Turn™ off-loads the patient's weight while the rounded shape and turning handles give caregivers the control to easily and safely reposition the patient.

EZ-Turn™ provides patient comfort and security and can help prevent injury for both the patient and caregiver.

### FEATURES

- Core is constructed from high-density CertiPUR-US® certified, PPF™ foam
- Foam is free from formaldehyde and toxic heavy metals
- Durable and resilient for years of use
- Covered in a durable heavy-duty medical grade vinyl
- Double stitched to improve performance and longevity
- Heavy-duty double stitched handles for excellent grasp and perfect control
- Cleans easily
- Made in the USA

# EZ-TURN™

## SPECIFICATIONS

Model BC-EZTA	18" w x 16" h x 5" d (leg opening 4.5" w)
Model BC-EZTA-BAR	22" w x 18.5" h x 7" d (leg opening 6" w)
Cover:	Medical Grade Vinyl, Double Stitched
Core:	Certipur certified, PPF™ foam



Model Shown BC-EZTA

### TURN EASY, TURN SAFELY

With the patient lying on their back in bed:

#### STEP 1

Position the EZ-Turn™ flat on the bed with the leg supports facing down toward the foot of the bed. Position the patient's left and right leg over each leg opening.



#### STEP 2

Stand the EZ-Turn™ upright so the patient's legs fit through the leg opening.



#### STEP 3

With the patient's legs properly positioned, push or pull the side handles to easily roll the patient.



### WARRANTY

6 months limited warranty on foam core



**Blue Chip**

Medical Products, Inc.

7-11 Suffern Place, Suffern NY 10901

Tel: (845) 369-7535 • Fax: (845) 369-7633 • Toll Free: (800) 795-6115

[www.bluechipmedical.com](http://www.bluechipmedical.com)

*"Setting the Standard"*

©Blue Chip Medical Products, Inc. 2021

Alternating Pressure Mattresses  
Alternating Pressure Pads & Pumps  
Low Air-Loss Mattress Systems  
Lateral Rotation Mattress Systems  
Gel Mattresses  
Gel Mattress Overlays  
Static-Air Mattress Overlays  
Water Mattress Overlays  
Wheelchair Cushions  
Patient Lift Slings  
Side Rail Bumpers  
Cart Covers  
OR Replacement Table Pads  
Replacement Stretcher Pads  
Cubicle Curtains  
Hospital Pillows